



② SINGLE USE STERILE R

ADDITIONAL MATERIALS NEEDED:

For Papanicolaou Technique:

1. Vaginal Speculum
2. Two glass slides.
3. Fixing solution (95% ethyl alcohol preferred; spray fix adequate).

For Liquid Based Cytology:

1. Vaginal Speculum
2. Preservative Fluid

PRIOR TO PROCEDURE

1. Obtain patient history and record.
2. Open fixative and label slides OR Label preservative fluid

CONTRAINDICATIONS

Usage is contraindicated in pregnancy and pelvic inflammatory diseases or when severe cervical stenosis exists.


SIDE EFFECTS

1. Patient may experience mild discomfort.
2. Warn patient that spotting may occur.


PRECAUTIONS:

1. Sterility is only guaranteed if packaging is intact.
2. For single use only.
3. If performing in conjunction with cervical smear, take endometrial sample first.
4. Knowledge of the uterine position is required before sampling.


DIRECTIONS

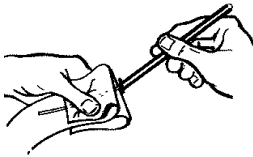


1. With sampler in sterile package, shape sampler shaft to conform to uterine position, if required.

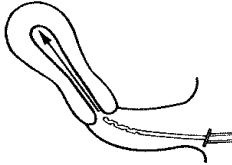

2. Gently insert sampler tip through cervical os. AVOID SCRAPING ENDOCERVIX.

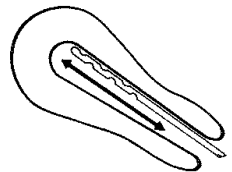

3. When fully inserted, collect sample by scraping the anterior endometrial surface three to five times. Repeat on posterior endometrial surface.

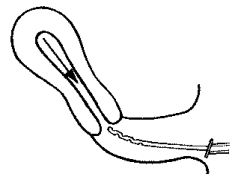


4. Promptly withdraw sampler, remove straight from cervical os. AVOID SCRAPING ENDOCERVIX.



The sample can then either be analyzed using the PAPANICOLAOU TECHNIQUE or by LIQUID BASED CYTOLOGY (LBC).

5. PAPANICOLAOU TECHNIQUE

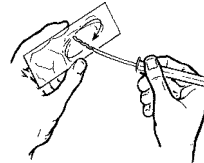

- 5.1. Place anterior collectors on surface of slide and spread sample evenly breaking up particles. Repeat with posterior collectors on second slide.



- 5.2. FIX EACH SLIDE IMMEDIATELY. Essential for accurate interpretation.

6. LIQUID BASED CYTOLOGY (LBC)



- 6.1 The sampler may be rinsed in the preservative fluid before disconnecting the head, to assist in dislodging sampled material.



If required, use a sharp pair of scissors, to cut off the sampling end of the Endo-pap curette and insert the end into the collection vial. Do not touch this end of the device. Discard the handle of the sampling device.

- 6.2. Cap the vial tightly and send the specimen containing the head of the sampling device, with appropriate paperwork, to the laboratory.

NAME AND BUSINESS ADDRESS OF THE HOLDER OF THE CERTIFICATE OF REGISTRATION:

Pharmaceutical Enterprises (Pty) Ltd,
Howard Studios, Howard Drive, Pinelands, 7405
P. O. Box 201, Howard Place, Cape 7450, R.S.A.
Tel: +2721 531-1341, Fax: +2721 531-2692

EC REP

Pharm-Ent Limited, 44, Beechwood Drive, Crewkerne.
Somerset TA18 7BY United Kingdom.
Tel/Fax No -44-1460271288

DISCLAIMER

PHARMACEUTICAL ENTERPRISES (PTY) LTD WILL NOT BE HELD RESPONSIBLE FOR MISS / POOR APPLICATION OF THE DEVICE / COMPLICATIONS ARISING FROM THIS.



0120

YENDCARD REV 1